



ACRO™ RSV Rapid Test Cassette

is a rapid chromatographic immunoassay for the qualitative detection of Respiratory Syncytial Virus antigen in nasopharyngeal swab or nasal aspirate specimens.

Features & Benefits

- Results in 15 minutes
- Easy to use
- Sample types:
 - ▶ Nasopharyngeal swab
 - ▶ Nasal aspirate specimens


Benefits of Rapid Diagnosis

- Guide appropriate therapy
- Avoid unnecessary antibiotic use
- Reduce cost of treatment

Sample	Sensitivity	Specificity	Accuracy
Nasopharyngeal swab	92.7%	98.0%	95.6%
Nasal aspirate	92.6%	98.5%	96.0%

See more about ACRO™ RSV performance in Package insert.


Direction for use

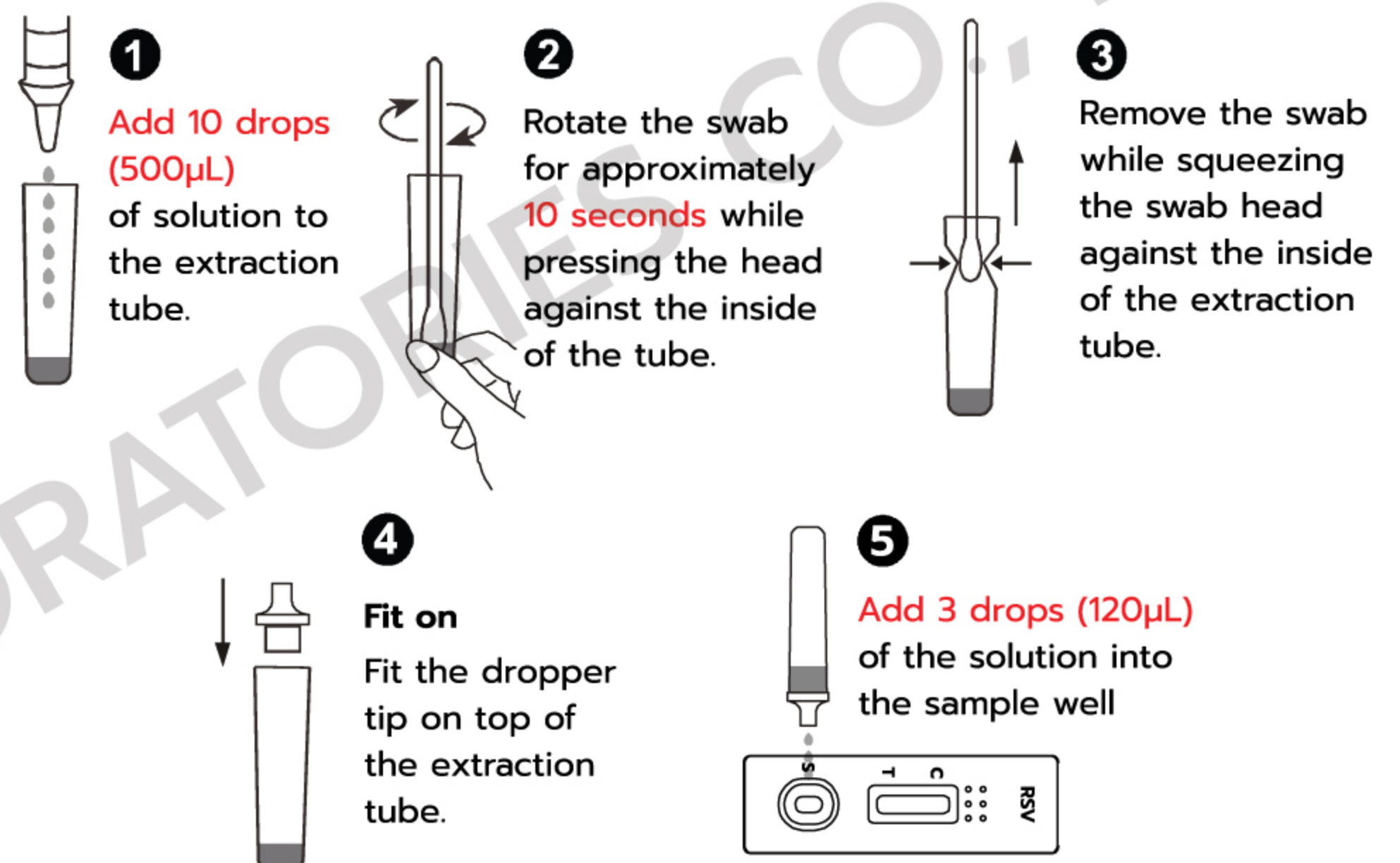

Read the result at 15 minutes

Note: Do not read result after 20 minute

Interpretation of Results

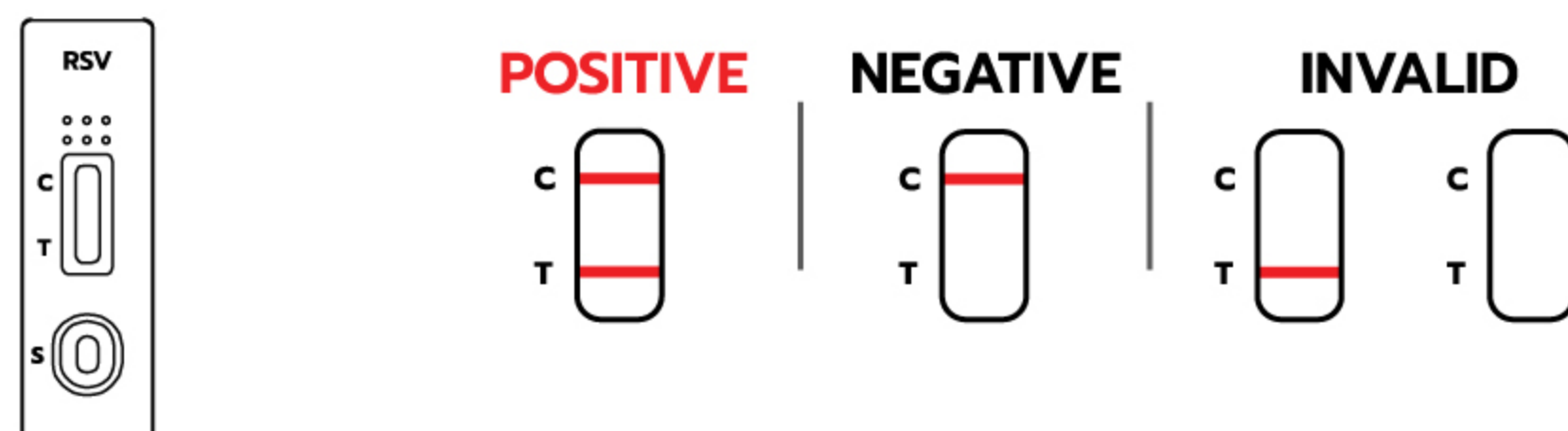

POSITIVE: Two distinct colored lines appear.

NEGATIVE: One colored line appears in the control line region (C).

INVALID: No line appears in the control line region (C).