



# ACRO hCG Pregnancy Rapid Test Dipstick (Urine)

## ACRO hCG Pregnancy Rapid Test Dipstick (Urine)

is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin in urine to aid in the early detection of pregnancy.



### Features & Benefits

- Can detect HCG at a concentration of 25mIU/ml or greater.
- Specimen type : First morning urine
- Result in 3 minutes
- Easy to use


ACRO hCG Pregnancy Rapid Test Dipstick (Urine)

Sensitivity	Specificity	Accuracy
>99.9%	>99.9%	>99.9%

\*compared with Other hCG Rapid Test

### Direction for use

1. Immerse the test dipstick vertically in the urine specimen for at least 15 seconds. Do not pass the maximum line (MAX) on the test dipstick when immersing the dipstick.
2. Read the result at 3 minutes. (Do not interpret the result after 10 minutes)

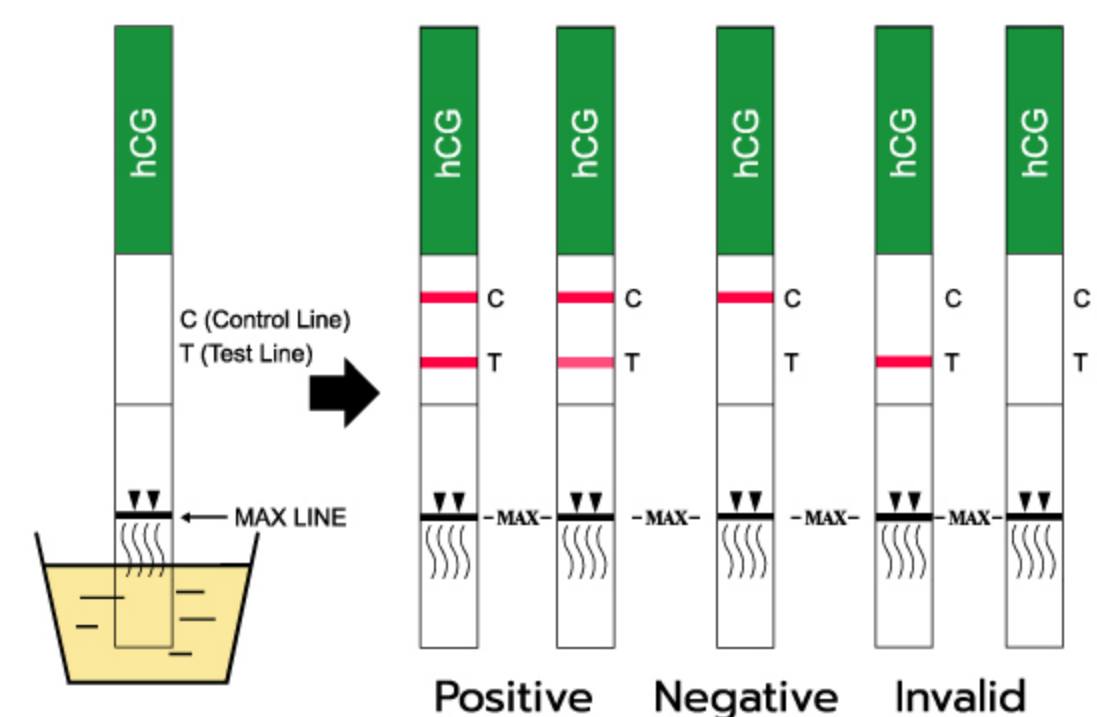

Positive Negative Invalid  
Read the result at 3 minutes.

### Interpretation of Results

- POSITIVE:** Two distinct colored lines appear. One line may be lighter than the other; they do not have to match. This means that you are probably pregnant.
- NEGATIVE:** One colored line appears in the control line region (C). This means that you are probably not pregnant.
- INVALID:** The result is invalid if no colored line appears in the control line region (C), even if a line appears in the test line region (T). You should repeat the test with a new test dipstick.