AirMini

คุณสมบัติ/Specifications

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AirMini

Specifications

- Automatic positive pressure air compressor (Auto CPAP) can set the pressure in the range of 4-20 cmH2O.
- There is a SmartStart system for automatic machine operation. When putting on/taking off the mask **If used with the P10 mask, there is a chance that the machine may not stop automatically.** Because the machine will stop when the high pressure is released more than 105 cmH2O.
- There is a function to help relieve pressure while exhaling, Expiratory Pressure Relief (EPR) up to 3 cmH2O.
- There is a symbol showing the tightness of wearing a mask (Mask Seal).
- Supports wireless communication (Wireless module) via the 3G/4G telephone network and connects to the phone via Bluetooth.
- CPAP machine size: height 136 mm x length 84 mm x width 52 mm
- CPAP machine weight 300 grams
- The operating sound level of the machine is 26.6 **decibels, but there may be a loud hissing sound from the pipe that disturbs sleep. Before buying, you should study or try before making a decision.**
- Ramp Time Delay timer system from off, 5 to 45 minutes or automatic.

Mode

- CPAP Constant pressure release mode
- AutoSet Automatic pressure release mode where the machine has algorithm that calculates from 3 breaths if In the event of apnea or light breathing, the machine will Release treatment pressure
- AutoSet FH Automatic pressure release mode where the machine has algorithm that calculates from 1 breath if In the event of apnea or light breathing, the machine will Release treatment pressure By gradually releasing the pressure The pressure will change gradually and smoothly. to prevent waking during the night

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- Auto CPAP
- AirMini pipe set, diameter 15 mm. The AirMini pipe set is as follows.
 - P10 AirMini Mask Pack HumidX, HumidX Plus
 - N20 AirMini Mask Pack HumidX, HumidX Plus
 - N30 AirMini Mask Pack HumidX, HumidX Plus
 - F20 AirMini Mask Pack Libu HumidX, HumidX Plus
- Adapter Power Supply Unit (Support Input range 110 - 240V, 20W)
- Travel bag
- Filter Standard
- User Manual EN/TH


Installation

- Connect the CPAP machine to the AirMini application with Bluetooth.
- Place the device on a stable, level surface. or use a bed mount
- Insert the power connector into the back of the device. Connect one end of the wire to the power supply unit. and the other end into an electrical outlet.
- Attach the air hose securely to the vent on the back of the device.
- Attach the free end of the air hose to the assembled mask tightly.
- Put the humidifier in the bulb.
- Press Start/Stop to begin use or breathe in and out when opening the Auto Start menu.

Reading results on the machine

- Access the sleep report at the AirMini application.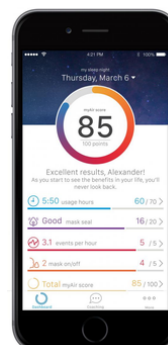

Note: Read the warnings on the label and medical device documentation.