Mode

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
Mode

- CPAP Constant pressure release mode
- AutoSet FH Automatic pressure release mode where the machine has algorithm that calculates from 1 breath if in the event of apnea or light breathing, the machine will Release treatment pressure By gradually releasing the pressure The pressure will change gradually and smoothly. to prevent waking during the night

AirSense 10 AutoSet For Her

Specifications

- Automatic positive pressure air compressor (Auto CPAP) can set the pressure in the range of 4-20 cmH2O.
- There is a SmartStart system for automatic machine operation. When putting on/taking off the mask
- Color LED screen, automatically adjusts screen brightness according to the light inside the room
- There is a function to help relieve pressure while exhaling, Expiratory Pressure Relief (EPR) up to 3 cmH2O.
- There is a symbol showing the tightness of wearing a mask (Mask Seal).
- Sleep report: Usage, Leak, Pressure, AI, HI, AHI, Cheyne–Stokes respiration, Hypopnoea, Central, Obstructive)
- Supports wireless communication (Wireless module) via 3G/4G telephone networks.
- CPAP machine size: height 116 mm x length 255 mm x width 150 mm
- CPAP machine weight 1284 grams
- The operating sound level of the machine is 26.6 decibels.
- The humidifier comes with a CPAP (HumidAir Cleanable) machine.
- Ramp Time Delay timer system from off, 5 to 45 minutes, or automatic.

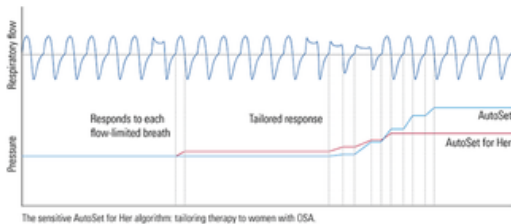Received equipment

- Auto CPAP machine with built-in humidifier
- HumidAir Cleanable
- ClimateLine
- Adapter Power Supply Unit (Support Input range 110 - 240V, 90W)
- Travel bag
- Filter Standard
- SD Card
- User Manual EN/TH

Installation

- Place the device on a stable, level surface.
- Insert the power connector into the back of the device. Connect one end of the wire to the power supply unit. and the other end into an electrical outlet.
- Attach the air hose securely to the vent on the back of the device.
- Attach the free end of the air hose to the assembled mask tightly.
- Put distilled medical water in the water bottle and insert the unit tightly.
- Press Start/Stop to begin use or breathe in and out when opening the Auto Start menu.

Reading results on the machine

- Access your sleep report at the Sleep Report menu.

Note: Read the warnings on the label and medical device documentation.