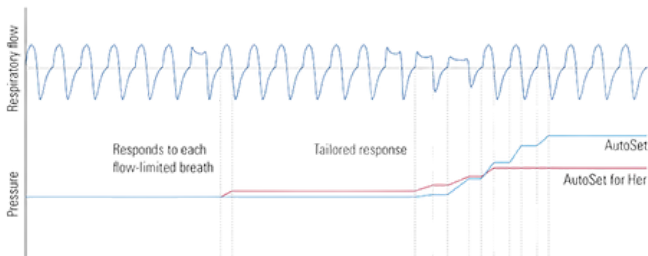
AirSense 10 AutoSet

คุณสมบัติ/Specifications

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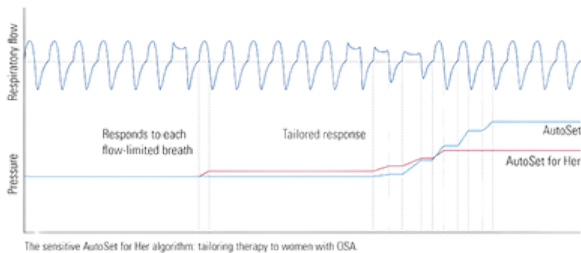
AirSense 10 AutoSet

Specifications

- Automatic positive pressure air compressor (Auto CPAP) can set the pressure in the range of 4-20 cmH2O.
- There is a SmartStart system for automatic machine operation. When putting on/taking off the mask
- Color LED screen, automatically adjusts screen brightness. according to the light inside the room
- There is a function to help relieve pressure while exhaling, Expiratory Pressure Relief (EPR) up to 3 cmH2O.
- There is a symbol showing the tightness of wearing a mask (Mask Seal).
- Sleep report: Usage, Leak, Pressure, AI, HI, AHI, Cheyne-Stokes respiration, Hypopnoea, Central, Obstructive)
- Supports wireless communication (Wireless module) via 3G/4G telephone networks.
- CPAP machine size: height 116 mm x length 255 mm x width 150 mm
- CPAP machine weight 1284 grams
- The operating sound level of the machine is 26.6 decibels.
- The humidifier comes with a CPAP (HumidAir Cleanable) machine.
- Ramp Time Delay timer system from off, 5 to 45 minutes or automatic.

The sensitive AutoSet for Her algorithm: tailoring therapy to women with USA.

Mode

- CPAP Constant pressure release mode
- AutoSet Automatic pressure release mode where the machine has algorithm that calculates from 3 breaths if it occurs In the event of apnea or light breathing, the machine will Release treatment pressure

Received equipment

- Auto CPAP , (HumidAir Cleanable)
- ClimateLine
- Adapter Power Supply Unit (Support Input range 110 - 240V, 90W)
- Travel bag
- Filter Standard
- SD Card
- (User Manual EN/TH)

Reading results on the machine

- Access your sleep report at the Sleep Report menu.

Installation

- Place the device on a stable, level surface.
- Insert the power connector into the back of the device. Connect one end of the wire to the power supply unit. and the other end into an electrical outlet.
- Attach the air hose securely to the vent on the back of the device.
- Attach the free end of the air hose to the assembled mask tightly.
- Put medical distilled water in the water bottle and insert the unit tightly.
- Press Start/Stop to begin use or breathe in and out when opening the Auto Start menu

Note: Read the warnings on the label and medical device documentation.